

IPAC EQUIPMENT CLEANING AND DISINFECTING MANUAL

September 2024

Adapted with permission from Vancouver Coastal Health

Interior Health would like to recognize and acknowledge the traditional, ancestral, and unceded territories of the Dākelh Dené, Ktunaxa, Nlaka'pamux, Secwépemc, St'át'imc, syilx, and Tšilhqot'in Nations where we live, learn, collaborate, and work together.

A **PRINTED** copy of this manual may not be the most recent version.

Contents

| | |
|--|----|
| Purpose..... | 1 |
| Definitions | 1 |
| General Principles | 2 |
| Designation of Cleaning and Disinfecting Responsibilities..... | 4 |
| Cleaning and Disinfection Products..... | 7 |
| How to Clean and Disinfect Equipment | 8 |
| Equipment Washers and Disposal Systems..... | 9 |
| Index | 10 |

Purpose

To provide guidance on roles, responsibilities, and standard principles associated with low-level cleaning and disinfection of medical equipment and devices.

Definitions

Cleaning

The physical removal of foreign material e.g., dusts, soil, organic material such as blood, secretions, excretions, and microorganisms using mechanical and/or chemical means. Cleaning physically removes rather than kills microorganisms.

Disinfectant

A product that is used on surfaces or medical equipment/devices which results in disinfection of the surface or equipment/device. Disinfectants are applied only to inanimate objects. Some products combine a cleaner with a disinfectant.

Manufacturer's instructions for use (MIFU)

Cleaning and disinfection procedures shall be based on device manufacturer's instructions.

Personal Protective Equipment (PPE)

Specialized clothing or equipment used by personnel to provide a barrier or shield to prevent exposure to potential infectious micro-organisms, and exposures to chemicals or physical hazards.

Ready-to-use (RTU)

Refers to pre-moistened disinfectant wipes which may be used for point-of-care cleaning and disinfection of non-critical patient medical devices according to the manufacturer's instructions.

Wet Contact time

The time required for surfaces to remain wet and air-dry following disinfection with a [RTU](#) wipe or cleaning/disinfecting cloth, as indicated on the [MIFU](#) or product label.

General Principles

1. Cleaning and disinfection of medical equipment/devices is a shared responsibility between multiple departments and health care providers.
2. The department manager or designate should establish a schedule for routine cleaning and disinfection of non-critical equipment to ensure adherence to [MIFU's](#)
 - At least annually.
 - With new implementation of equipment or cleaning procedures.
3. Where possible, dedicate equipment for a single patient (i.e., commodes, vital signs machines) especially when the patient is on Additional Precautions.
4. Equipment shared between patients must be cleaned and disinfected according to the [MIFU](#):
 - Between each use.
 - At the point of care (patients room/bed space etc.).
 - Before storage.
 - Before sending for repair.
 - When visibly soiled.
5. If an electronic item cannot be adequately cleaned and disinfected and will be accessed in a patient room or touched by patients, a cleanable or disposable cover shall be used.
6. Clean and disinfect grossly soiled equipment immediately to avoid contamination of the environment and damage to equipment from blood or body fluids drying on the surfaces.
7. When cleaning and disinfecting equipment, avoid splash contamination of nearby furnishings, linens, carts, and other clean items.
8. Designate a location or space for clean equipment storage. Clean storage rooms, service rooms, or utility rooms shall have a minimum two meters (Six feet) separation from dirty equipment and water sources.
9. Use hospital approved products for point of care cleaning and disinfection of equipment. Keep [RTU](#) wipes wet; discard if they become dry. Follow instructions on the product label for appropriate PPE and [wet contact time](#).
10. Contaminated/dirty equipment and beds must NOT be moved throughout the hospital.

11. Equipment that is cleaned and disinfected will be tagged with a green “**I am clean**” label or green tape, otherwise they are to be considered contaminated. EVS and frontline staff should follow the same process - “**I am clean**” label or green tape. After cleaning and disinfection, some equipment will be placed in the designated clean storage area.
12. Always remove the green “**I am clean**” label or green tape just before the equipment is used on a patient. Equipment going for repair to Biomed or Facilities Maintenance must be cleaned and disinfected and tagged with a green “**I am clean**” label or green tape.
13. Clean and contaminated/dirty equipment must be stored separately with a minimum of two meters separation.
14. Medical equipment/devices labelled by the manufacturer “single patient use” may be cleaned and disinfected for re-use on the same patient but must not be used on other patients.
15. Medical equipment/devices labelled by the manufacturer “single use/disposable” are for single use only and must be discarded immediately following use.
16. Do not use tape or labels that leave sticky residue on medical equipment/devices (i.e., taping garbage bags to V/S monitor).
17. Report damaged medical equipment to manager or designate. Ensure the item is clean and disinfected before being sent for repair.
18. Staff owned/dedicated equipment (phones, tablets, stethoscopes, pagers etc.) must be cleaned and disinfected between patients, and immediately after use if in contact with the patient or the patients’ environment.
19. Nursing units (i.e., Intensive Care Unit and Emergency) may have their own cleaning and disinfecting protocols for certain specialty items. Always confirm what these protocols are before you start working in those locations.
20. Always refer to the [MIFU](#) for cleaning and disinfecting equipment.
21. For EVS best practices, refer to EVS [Standard Operating Procedures](#) for detailed cleaning and disinfection instructions.

Designation of Cleaning and Disinfecting Responsibilities




| WHO | WHAT | WHEN |
|---|--|--|
| <p>Health Care Provider (HCP)</p> <p>Includes all health care workers (doctors, nurses, respiratory therapists/aides, occupational therapy/physiotherapy (OT/PT), rehab assistants, diagnostic imaging technicians, lab and cardiac technicians, porters, etc.).</p> | <p>Clean and disinfect:</p> <ul style="list-style-type: none"> • Mobile equipment used between multiple patients. • Equipment dedicated to one patient. • Equipment attached to the patient (in use). | <ul style="list-style-type: none"> • Between patients and/or immediately after use. • Following contact with patient environment. • Daily (ward routine schedule). • When visibly soiled. • Upon patient discharge. • Prior to storage. • Prior to sending for repair. • After being repaired. |
| | <p>Clean and disinfect:</p> <ul style="list-style-type: none"> • Equipment owned or designated to staff for personal use (e.g., stethoscopes, badges, mobile phones, pagers, etc.). | <ul style="list-style-type: none"> • Between patients and/or immediately after use. • Following contact with patient/patient's environment. |

| WHO | WHAT | WHEN |
|--|--|--|
| Environmental Services Staff (EVS) | Clean and disinfect: <ul style="list-style-type: none"> Equipment in patient room not attached to the patient. | <ul style="list-style-type: none"> Daily during routine cleaning and disinfection. When visibly soiled. |
| | Clean and disinfect: <ul style="list-style-type: none"> Equipment in patient room upon discharge/transfer. | <ul style="list-style-type: none"> During discharge cleaning and disinfection (per Routine and Additional Precautions). Prior to storage. Prior to sending for repair. |
| | Clean and disinfect: <ul style="list-style-type: none"> Equipment placed in soiled utility rooms. Check soiled utility rooms to prevent overcrowding. | <ul style="list-style-type: none"> Check daily. |
| Managers and Supervisors (both HCP and EVS) | <ul style="list-style-type: none"> Ensure HCP and EVS have the training and supplies needed to clean and disinfect equipment. Ensure equipment cleaning and disinfection protocols are monitored. Ensure adherence to manufacturer's instructions for cleaning and disinfection, in addition to infection prevention and control standards. | <ul style="list-style-type: none"> Per orientation and training schedules. Ensure equipment cleaning and disinfection protocols are monitored at least annually. Ensure equipment cleaning protocols implemented for new equipment and/or procedures. |
| BioMed | Clean and disinfect: <ul style="list-style-type: none"> Equipment after repair/maintenance. | <ul style="list-style-type: none"> Before returning to unit or equipment depot. |

| WHO | WHAT | WHEN |
|---|--|---|
| <p>Equipment Depot (EQD) staff <i>*Where available</i></p> | <p>Clean and disinfect:</p> <ul style="list-style-type: none"> • All equipment designated for storage. • All equipment before returning to unit. | <ul style="list-style-type: none"> • Per EQD schedule. • Ensure equipment is cleaned and disinfected before transporting through clean/sterile storage areas, patient care areas or high traffic areas. |

Cleaning and Disinfection Products

At Interior Health, the following products are available for HCPs to clean and disinfect equipment. Please note this list does not include products utilized by Environmental Services for cleaning and disinfection:

| Product | When to Use | Wet Contact Time |
|--|--|--|
| Accelerated Hydrogen Peroxide wipes (AHP wipes - Accel INTERvention)  | These are to be used to clean and disinfect most surfaces on medical devices/equipment. | One minute. |
| Complete 6000 wipes  | These are to be used to clean and disinfect surfaces when a sporicidal product is required. | Five minutes. Rinse off food contact surfaces with water after wet contact time complete. |
| Alcohol wipes (Kimtech)  | These are to be used on all displays/screens where indicated. *Screens with plastic protectors can be cleaned and disinfected with AHP wipes* | No wet contact time. |

Keep container caps of wipes closed when removing wipes. Leaving the caps open allows the wipes to dry out and renders them ineffective for cleaning and disinfection.

HCP's should not access cleaning and disinfecting solutions and supplies that are dedicated for use by EVS.

How to Clean and Disinfect Equipment

Cleaning and Disinfection Require a Two-Step Process:

Step 1: Clean

- Clean to remove any foreign matter from the device using friction (rub/scrub motion).
- Clean from cleanest area to dirtiest area.
- Use as many wipes/cloths as needed until wipe comes away clean.
- Follow instructions for use on the product label for further information.

Step 2: Disinfect

- Use fresh wipes/cloth to disinfect again using a friction (rub/scrub) motion.
- Allow surfaces to remain wet for the necessary 'contact time' followed by air-drying as written on the product label.
- Avoid electronic connectors to prevent malfunction.

Step 3: Rinse

- To avoid damage to screens from cleaning and disinfecting residue, wipe with plain water using well squeezed dampened cloth or a 70% alcohol wipe after air-drying is complete.

Step 4: Label

- Attach a green "I am clean" sticker to the equipment before returning to clean storage/area.

Equipment Washers and Disposal Systems

Macerators (Vernacare)

Reduces **disposable** products to a watery slurry for drainage into existing facilities' plumbing.

- Environmental Services to wipe down external surfaces using hospital approved disinfectant daily and as required.
- Follow manufacturer directions for number of single use containers to use at one time.
- Manager or designate to review loading procedures with staff at time of hire and with introduction of new processes or products.

Washer/Disinfector (Meiko)

Intended for emptying, flushing, cleaning, and thermally disinfecting by steam, human waste containers intended for **re-use** such as bed pans/supports, urine bottles, measuring containers etc.

- Environmental Services to wipe down external surfaces using hospital approved disinfectant daily and as required.
- **Store clean and disinfected items on their side or upside down** to facilitate drainage and complete drying.
- Establish a schedule for routine cleaning of containers (i.e., washbasins).
- Manager or designate to review loading procedures with staff at time of hire and with introduction of new processes or product.

Wheelchair Washers





- Follow the MIFU re: proper use.
- Environmental Services to wipe down external surfaces using hospital approved disinfectant daily and as required.
- Establish a schedule for routine cleaning of wheelchairs.
- Manager or designate to review loading procedures with staff at time of hire and with introduction of new processes or product.

Index




Not all equipment is specifically itemized in this document. If you have questions about who is responsible for cleaning an item that is not listed, please contact your

- | | |
|--|---|
| 1. Analyzer, oxygen | 31. Flowmeter, oxygen/air |
| 2. Backboards/sliders | 32. Fridge, medication |
| 3. Basin, bath, or wash | 33. Fridge, patient nourishment |
| 4. Bassinette | 34. Fridge, staff |
| 5. Bed/chair check alarm | 35. GlideScope |
| 6. Beds and attachments | 36. Glucometer, docking station and carrying case |
| 7. Bed, Billie/Blanket/UV Light | 37. HoverMatt |
| 8. Bedpan | 38. Incubator, Neonatal |
| 9. Bladder scanner (various models) | 39. Infuser, rapid (Rapid infuser) |
| 10. Bronchoscope monitor | 40. Lamp, portable (examination) |
| 11. Calf compressor (SCD) | 41. Laryngoscope, video (various models) |
| 12. Call bell | 42. Lead apron |
| 13. Cart, bedside/supply (various models) | 43. Lift, patient, ceiling mounted |
| 14. Cart, bronchoscopy | 44. Lift, patient, portable (with sling) |
| 15. Cart, crash cart | 45. Mattress (including specialty) |
| 16. Cart, dirty laundry | 46. Medication dispensing system (various models) |
| 17. Cart, isolation | 47. Microscope |
| 18. Cart, medicine (med cart) | 48. Mobility aides (walker, cane, crutches) |
| 19. Chairs, generic | 49. Monitor, bedside (various models) |
| 20. Chairs, Geri and Broda, Wheelchair and Transport | 50. Monitor, central patient |
| 21. Commodes | 51. Monitor, ETCO2 (various models) |
| 22. Computer | 52. Monitor, fetal |
| 23. Cooling unit, patient | 53. Monitor, transport (various models) |
| 24. Defibrillator (various models) | 54. Nebulizer (portable air compressor) |
| 25. Dividers | 55. Ophthalmoscope/Otoscope |
| 26. Doppler, handheld (various models) | 56. Oxygen concentrator (various models) |
| 27. Doppler, portable (various models) | 57. Oxygen tank |
| 28. ECG machine | 58. Pacemaker (external) |
| 29. Electrocautery (ESU) | 59. Parallel bars |
| 30. Equipment going out for/returning after repair | 60. Physio Stairs |
| | 61. Pillows |




62. [Pneumatic tube system, carrier station](#)
63. [Pneumatic tube system, cylinder](#)
64. [Pole, IV \(single and multi\)](#)
65. [Pressure bag](#)
66. [Pressure manometer](#)
67. [Printer/fax machine](#)
68. [Prismaflex](#)
69. [Probe, TEE \(and stand\)](#)
70. [Pulse Oximeter](#)
71. [Pump, breast](#)
72. [Pump \(Infusion, Feeding, PCA, epidural\)](#)
73. [Radio \(hospital owned\)](#)
74. [Rehab equipment](#)
75. [Restraints](#)
76. [Scale, portable](#)
77. [Scope Monitor \(Bronchoscope etc.\)](#)
78. [Seizure pad \(fall mats\)](#)
79. [Sliding board](#)
80. [Sling](#)
81. [Slit lamp](#)
82. [Stretchers \(including mattress\)](#)
83. [Stethoscope](#)
84. [Suction, portable](#)
85. [Suction, regulator wall mount](#)
86. [Table, bedside](#)
87. [Table, examination](#)
88. [Table, procedure](#)
89. [Telemetry pack](#)
90. [Telephone](#)
91. [Television \(ceiling mounted and remote\)](#)
92. [Thermometer, all types](#)
93. [Toilet seat, raised](#)
94. [Tonometer \(ED\)](#)
95. [Tonopen \(ED\)](#)
96. [Toys](#)
97. [Tubs](#)
98. [Ultrasound, diagnostic](#)
99. [Urine Analyzer \(ED\)](#)
100. [V.A.C. unit - Wound \(IH Owned\)](#)
101. [V.A.C unit - Wound \(Vendor Owned\)](#)
102. [Ventilator, various models \(Bipap, CPAP, Optiflow\)](#)
103. [Vital Signs Monitor, mobile](#)
104. [Warmer, blanket](#)
105. [Warmer, blood/IV fluid](#)
106. [Warmer, infant and neonatal](#)
107. [Warmer, patient \(Bair Hugger\)](#)
108. [Warmer, radiant overhead](#)
109. [Warmer, Sage Wipe](#)
110. [Water/Ice Machine](#)
111. [Wedges](#)
112. [Wheelchairs and cushions](#)
113. [Workstation on wheels \(WOW, e.g., computers\)](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|-----------------------------------|--|---|
| <p>1. Analyzer, oxygen</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage</p> |
| <p>2. Backboards/sliders</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>3. Basin, bath, or wash</p>  | <p>Soiled utility room</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Use washer/disinfector or send to reprocessing area.</p> |
| <p>4. Bassinette</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> High touch surfaces daily. Minimum weekly when in use. When soiled. | <p>AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> <ul style="list-style-type: none"> At discharge. After each use. | <p>Follow EVS best practices.</p> <p>Attach green “I am clean” label and return to clean storage.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|-----------------------------------|--|--|
| <p>5. Bed/chair check alarm</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • Minimum weekly when in use. • When soiled. | <p>AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> <ul style="list-style-type: none"> • At discharge. • After each use. | <p>Follow EVS best practices. Attach green “I am clean” label and return to clean storage.</p> |
| <p>6. Beds and attachments</p>  <p>Also see “Mattress”</p> | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • High touch surfaces daily. • Minimum weekly when in use. • When soiled. <p>EVS</p> <ul style="list-style-type: none"> • At discharge. • After each use. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> |
| <p>7. Bed, Billie/blanket/UV light</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • High touch surfaces daily. • Minimum weekly when in use. • When soiled. | <p>AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> <ul style="list-style-type: none"> • At discharge. • After each use. | <p>Follow EVS best practices. Attach green “I am clean” label and return to clean storage.</p> |

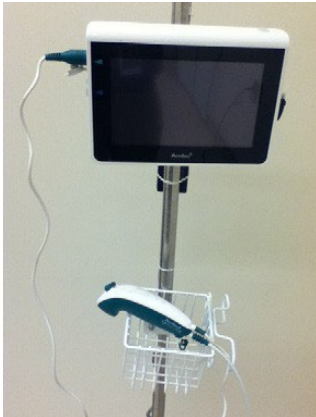


[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|-----------------------------------|--|---|
| <p>8. Bedpan</p>  | <p>Soiled utility room</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Use washer/disinfector or send to reprocessing area.</p> |
| <p>9. Bladder scanner (various models)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>AHP wipes</p> <p>Clean and disinfect display, probe, probe cable, and all high touch points.</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>10. Bronchoscope monitor</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|-----------------------------------|---|---|
| <p>11. Calf compressor (SCD)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • High touch surfaces daily. • Minimum weekly when in use. • When soiled. | <p>AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> <ul style="list-style-type: none"> • At discharge. • After each use. | <p>Follow EVS best practices.</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>12. Call bell</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> • High touch surfaces daily. • When soiled. • At discharge. • After each use. | <p>Follow EVS best practices.</p> |
| <p>13. Cart, bedside/supply (various models)</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> • High touch surfaces daily. • When soiled. • At discharge. • After each use. | <p>Follow EVS best practices.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|-----------------------------------|---|--|
| <p>14. Cart, bronchoscopy</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>15. Cart, crash cart</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. • Weekly. | <p>AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>16. Cart, dirty laundry</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> • High touch surfaces. • Daily. | <p>Follow EVS best practices.</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> <ul style="list-style-type: none"> • Minimum weekly. • When soiled. • Prior to storing after use. | <p>Follow EVS best practices.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|--------------------------------|---|---|
| <p>17. Cart, isolation</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> • High touch surfaces daily. <p>HCP</p> <ul style="list-style-type: none"> • When soiled. • After isolation precautions discontinued. • Prior to storing after use. | <p>Follow EVS best practices</p> <p>AHP wipes</p> <p>Empty cart, clean and disinfect, and replenish cart.</p> <p>Attach green “I am clean label” and return to clean storage.</p> |
| <p>18. Cart, medicine</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • High touch surfaces daily. • Minimum weekly including drawers. • When soiled. | <p>AHP wipes</p> |
| <p>19. Chairs, generic</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> • High touch surfaces daily. • When soiled. • At discharge. | <p>Follow EVS best practices.</p> |





[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|---|--|
| <p>20. Chairs, Geri, Broda, Wheelchair, and Transport</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> • High touch surfaces daily. • Minimum weekly when in use. • When soiled. • After each use • Prior to storing after use. | <p>AHP wipes</p> <p>Attach green “I am clean label” and return to clean storage.</p> |
| <p>21. Commodes</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> • After each use – Empty and clean visible soiling and high touch surfaces. <p>EVS</p> <ul style="list-style-type: none"> • At discharge. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> <p>Attach green “I am clean label” and return to clean storage.</p> |
| <p>22. Computer</p>  <p>Mobile computers – See “Workstation on Wheels”</p> | Point of care | <p>HCP</p> <ul style="list-style-type: none"> • Daily. • When soiled. • After direct or indirect patient contact. | <p>Include display/screen, keyboard, and mouse.</p> <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> |



[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|-----------------------------------|--|--|
| <p>23. Cooling unit, patient</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> High touch surfaces daily. Minimum weekly when in use. When soiled. | <p>AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> <ul style="list-style-type: none"> At discharge. After each use. | <p>Follow EVS best practices.</p> <p>Attach green “I am clean label” and return to clean storage.</p> |
| <p>24. Defibrillator (various models)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>25. Dividers</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> After each use. | <p>Follow EVS best practices.</p> <p>Attach green “I am clean” label and return to clean storage.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|--|--|
| <p>26. Doppler, handheld (various models)</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>27. Doppler, portable (various models)</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>28. ECG machine</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>29. Electrocautery (ESU)</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>AHP wipes</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|---|---|
| <p>30. Equipment going out for/returning after repair</p>  | <p>See process</p> | <p>See process</p> | <p>Process</p> <ol style="list-style-type: none"> 1. Equipment is cleaned and disinfected as outlined in this document. 2. Facilities Maintenance or Biomed request is generated. 3. Equipment is repaired. 4. After repair, Facilities Maintenance or Biomed cleans equipment as per their best practice. 5. Equipment is again cleaned and disinfected as outlined in this document prior to use. 6. Green is clean sticker is applied after each cleaning and before use. |
| <p>31. Flowmeter, oxygen/air</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> • High touch surfaces daily. • When soiled. • At discharge. • Between patients. | <p>Follow EVS best practices.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|---------------------------------------|--|--|
| <p>32. Fridge, medication</p>  | <p>Point of care</p> | <p>Exterior surfaces and handle: EVS</p> <ul style="list-style-type: none"> • Daily. • When soiled. <p>Interior surfaces: HCP</p> <ul style="list-style-type: none"> • Monthly. • When soiled. | <p>Follow EVS best practices.</p> <p>AHP wipes</p> |
| <p>33. Fridge, patient nourishment</p>  | <p>Point of care</p> | <p>Exterior surfaces and handle: EVS</p> <ul style="list-style-type: none"> • Daily. <p>Interior surfaces: HCP/Food Services</p> <ul style="list-style-type: none"> • Monthly. • When soiled. | <p>Follow EVS best practices.</p> <p>HCP empties fridge and coordinates cleaning with Food Services.</p> |
| <p>34. Fridge, staff</p>  | <p>Lunchroom/ staff lounge</p> | <p>Exterior surfaces and handle: EVS</p> <ul style="list-style-type: none"> • Daily. • When soiled. <p>Interior surfaces: HCP/EVS</p> <ul style="list-style-type: none"> • Monthly. • When soiled. | <p>Follow EVS best practices.</p> <p>HCP empties fridge and coordinates cleaning with EVS.</p> |




[Back to index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|--------------------------------|---|---|
| <p>35. GlideScope</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Remove disposables.</p> <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green “I am clean” sticker and return to clean storage.</p> |
| <p>36. Glucometer, Docking Station, and Carrying Case</p>  | <p>Point of care</p> | <p>Glucometer HCP</p> <ul style="list-style-type: none"> After each use. Daily. <p>Docking Station</p> <ul style="list-style-type: none"> Weekly. When visibly soiled. <p>Carrying Case</p> <ul style="list-style-type: none"> Monthly. When visibly soiled. | <p>Follow manufacturer’s instructions for cleaning and disinfection.</p> <p>Display/screen. Alcohol wipes</p> <p>All other parts AHP wipes</p> |
| <p>37. HoverMatt</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. <p>Discard single use only air mattresses.</p> <p>Multi-use mattresses can go in laundry hamper or call ‘the Porter’ to take it for laundering.</p> | <p>Display/screen. Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green “I am clean” sticker and return to clean storage.</p> |




[Back to index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|-----------------------------------|--|--|
| <p>38. Incubator, Neonatal</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • High touch surfaces daily. • Minimum weekly when in use. • When soiled. | <p>Follow manufacturer's instructions for cleaning and disinfection.</p> <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>HCP/MDR</p> <ul style="list-style-type: none"> • At discharge. • Between patients. | <p>Follow MIFU for cleaning and disinfection.</p> <p>Attach green "I am clean" label and return to clean storage.</p> |
| <p>39. Infuser, rapid (Rapid infuser)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use clean and place in soiled utility room. | <p>AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> | <p>Follow EVS best practices.</p> <p>Attach green "I am clean" label and return to clean storage.</p> |
| <p>40. Lamp, portable (examination)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use clean and place in soiled utility room. | <p>AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> | <p>Follow EVS best practices.</p> <p>Attach green "I am clean" label and return to clean storage.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|-----------------------------------|---|---|
| <p>41. Laryngoscope, video (various models)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use, remove disposables, place rigid stylets in soiled respiratory bin and send to MDRD. Clean laryngoscope and place in soiled utility room. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> | <p>Follow EVS best practices.</p> <p>Attach green "I am clean" label and return to clean storage.</p> |
| <p>42. Lead Apron</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. <p>EVS</p> <ul style="list-style-type: none"> Weekly. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> |
| <p>43. Lift, patient, ceiling mounted</p>  | <p>Point of care</p> | <p>Handles and remote-control unit:</p> <p>EVS</p> <ul style="list-style-type: none"> Daily. When soiled. At discharge. Between patients. <p>HCP</p> <ul style="list-style-type: none"> Between patients. | <p>Follow EVS best practices.</p> <p>AHP wipes</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|--------------------------------|---|---|
| <p>44. Lift, patient, portable (with sling)</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> • After each use. | <p>AHP wipes</p> <p>See slings below for cleaning and disinfection recommendations.</p> |
| <p>45. Mattress (including specialty)</p>  | Point of care | <p>EVS</p> <ul style="list-style-type: none"> • When soiled. • At discharge. • Between patients. • After 30 days, re-clean and attach a new green “I am clean” label. | <p>Follow EVS best practices.</p> <p>Do NOT place on floor after cleaning!</p> <p>If not in use, attach green “I am clean” label and return to clean storage.</p> <p>Place sheet on mattress before transporting.</p> |
| <p>46. Medication dispensing system (various models)</p>  | Point of care | <p>Exterior surfaces:</p> <p>EVS</p> <ul style="list-style-type: none"> • Daily. • When soiled. <p>Interior surfaces:</p> <p>HCP/Pharmacy Tech</p> <ul style="list-style-type: none"> • Monthly. • When soiled. | <p>Follow EVS best practices.</p> <p>AHP wipes</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|---|--|
| <p>47. Microscope</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. | <p>AHP wipes</p> |
| <p>48. Mobility aides (e.g., walker, cane, crutches)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • Weekly. • When soiled. • At discharge. • Between patients. | <p>AHP wipes</p> |
| <p>49. Monitor, bedside (various models)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • Daily. • When soiled. <p>EVS</p> <ul style="list-style-type: none"> • At discharge. • Between patients. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Follow EVS best practices.</p> |





[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|--------------------------------|---|--|
| <p>50. Monitor, central patient</p>  | <p>Point of care</p> | <p>HCP/Unit Clerk</p> <ul style="list-style-type: none"> Daily: Display/screen, keyboard, mouse. When soiled. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> |
| <p>51. Monitor, ETCO2 cartridge (various models)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>AHP wipes</p> |
| <p>52. Monitor, fetal</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|-----------------------------------|--|---|
| <p>Monitor, transport (various models)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage</p> |
| <p>54. Nebulizer (portable air compressor)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use clean and place in soiled utility room. | <p>AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> | <p>Follow EVS best practices.</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>55. Ophthalmoscope/Otoscope</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. <p>EVS</p> <ul style="list-style-type: none"> Housing and cables/handles on discharge. | <p>Alcohol wipes</p> <p>Follow EVS best practices.</p> |





[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|-----------------------------------|--|---|
| <p>56. Oxygen concentrator (various models)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • High touch surfaces daily. • Minimum weekly when in use. • When soiled. | <p>AHP wipes</p> |
| | <p>Soiled utility room</p> | <p>EVS</p> <ul style="list-style-type: none"> • At discharge. | <p>Follow EVS best practices.</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>57. Oxygen tank</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. | <p>AHP wipes</p> |
| <p>58. Pacemaker(external)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. | <p>AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>59. Parallel bars</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. <p>EVS</p> <ul style="list-style-type: none"> • Daily. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|--------------------------------|--|--|
| <p>60. Physio stairs</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. <p>EVS</p> <ul style="list-style-type: none"> Daily. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> |
| <p>61. Pillows</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> At discharge. Between patients. | <p>Follow EVS best practices.</p> |
| <p>62. Pneumatic tube system, carrier station</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> Daily. | <p>Follow EVS best practices.</p> |

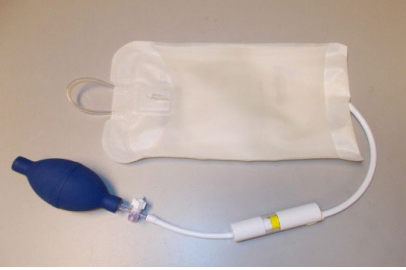



[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|--|--|
| <p>63. Pneumatic tube system, cylinder</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> When soiled. Quarterly. | AHP wipes |
| <p>64. Pole, IV (single and multi)</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> Daily. When soiled. Between patients. <p>EVS</p> <ul style="list-style-type: none"> At discharge. Between patients. After 30 days when not in use. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>65. Pressure bag</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> After each use. <p>EVS</p> <ul style="list-style-type: none"> At discharge. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> |
| <p>66. Pressure manometer</p>  | Point of care | <p>HCP</p> <ul style="list-style-type: none"> After each use. | AHP wipes |





[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|---------------------------------|--|--|
| <p>67. Printer/fax machine</p>  | <p>Various locations</p> | <p>HCP/Unit Clerk</p> <ul style="list-style-type: none"> Daily. | <p>AHP wipes</p> |
| <p>68. Prismaflex</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> |
| <p>69. Probe, TEE (and stand)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use clean and place in transport stand/shuttle and send to MDRD. | <p>AHP wipes</p> |





[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|--------------------------------|---|---|
| 70. Pulse Oximeter (portable)  | Point of care | HCP <ul style="list-style-type: none"> After each use. | AHP wipes |
| 71. Pump, breast  | Point of care | HCP <ul style="list-style-type: none"> After each use. | AHP wipes Attach green "I am clean" label and return to clean storage. |
| 72. Pump (infusion, feeding, PCA, epidural)  | Point of care | HCP <ul style="list-style-type: none"> Remove tubing set After each use. EVS <ul style="list-style-type: none"> At discharge After 30 days if not in use. | AHP wipes Follow EVS best practices. Attach green "I am clean" label and return to clean storage. |
| 73. Radio (hospital owned)  | Point of care | HCP <ul style="list-style-type: none"> After each use. EVS <ul style="list-style-type: none"> Daily. At discharge. | AHP wipes Follow EVS best practices. Attach green "I am clean" label and return to clean storage. |





[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|--|--|
| <p>74. Rehab equipment</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>75. Restraints</p>  | <p>Laundry</p> | <p>HCP</p> <ul style="list-style-type: none"> Laundry after each use. | <p>Restraints are placed in regular laundry hampers.</p> |
| <p>76. Scale, portable</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>77. Scope Monitor (Bronchoscope etc.)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |

[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|--|--|
| <p>78. Seizure pad (fall mats)</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> • At discharge. • When soiled. • After 30 days when not in use. | <p>Follow EVS best practices.</p> |
| <p>79. Sliding board</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. | <p>AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>80. Sling</p>  | <p>Laundry</p> | <p>HCP</p> <ul style="list-style-type: none"> • Launder between patient use, and when soiled. | <p>Slings are placed in regular laundry hampers or follow site specific laundering instructions.</p> |
| <p>81. Slit lamp</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. | <p>AHP wipes</p> |





[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|---|---|
| 82. Stretchers (including mattress)  | Point of care | HCP <ul style="list-style-type: none"> • After each use. EVS <ul style="list-style-type: none"> • At discharge. • Once weekly when not in use. | AHP wipes Follow EVS best practices. Attach green “I am clean” label and return to clean storage. |
| 83. Stethoscope  | Point of care | HCP <ul style="list-style-type: none"> • After each use. | AHP wipes |
| 84. Suction, portable  | Point of care | HCP <ul style="list-style-type: none"> • After each use • remove and replace all disposable parts. | AHP wipes Attach green “I am clean” label and return to clean storage. |
| 85. Suction, regulator wall mount  | Patient room | EVS <ul style="list-style-type: none"> • At discharge. | Follow EVS best practices. |





[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|--------------------------------|---|---|
| 86. Table, bedside  | Point of care | HCP <ul style="list-style-type: none"> As needed/when soiled. EVS <ul style="list-style-type: none"> Daily. On patient discharge. Between patients. | AHP wipes Follow EVS best practices. |
| 87. Table, examination  | Point of care | HCP <ul style="list-style-type: none"> After each use. EVS <ul style="list-style-type: none"> Daily. | AHP wipes Follow EVS best practices. |
| 88. Table, procedure  | Point of care | HCP <ul style="list-style-type: none"> After each use. | AHP wipes Attach green "I am clean" label and return to clean storage. |
| 89. Telemetry pack  | Point of care | HCP <ul style="list-style-type: none"> Daily when in use. When soiled. | AHP wipes Attach green "I am clean" label and return to clean storage. |





[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|---------------------------------|---|---|
| <p>90. Telephone</p>  | <p>Various locations</p> | <p>HCP/Unit clerk</p> <ul style="list-style-type: none"> Daily. | <p>AHP wipes</p> |
| <p>91. Television (ceiling mounted and remote)</p>   | <p>Patient room</p> | <p>EVS</p> <ul style="list-style-type: none"> Daily. At discharge. Between patients. | <p>Follow EVS best practices.</p> <p>Attach green "I am clean" label and return to clean storage.</p> |
| <p>92. Thermometer, all types</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. <p>EVS</p> <ul style="list-style-type: none"> Daily. At discharge. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|---|--|
| 93. Toilet seat, raised  | Point of care | EVS <ul style="list-style-type: none"> • Daily. • When soiled. • Multi-bed rooms every 4 hours. • At discharge. • 30 days of no use. | Follow EVS best practices. Attach green “I am clean” and return to clean storage. |
| 94. Tonometer (ED)  | Point of care | HCP <ul style="list-style-type: none"> • After each use. | Display screen Alcohol wipes All other parts AHP wipes |
| 95. Tonopen (ED)  | Point of care | HCP <ul style="list-style-type: none"> • After each use. | AHP wipes |
| 96. Toys  | Point of care | HCP <ul style="list-style-type: none"> • After each use. • When soiled. • Once weekly. | AHP wipes |




[Back to Index](#)
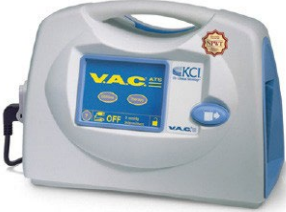
| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|---------------------------------------|--|---|
| <p>97. Tubs</p>  | <p>Tub room</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. <p>EVS</p> <ul style="list-style-type: none"> Exterior Daily. | <p>AHP wipes</p> <p>Follow EVS best practices</p> |
| <p>98. Ultrasound, diagnostic</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>Display/screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Attach green "I am clean" label and return to clean storage.</p> |
| <p>99. Urine analyzer (ED)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> Daily. | <p>Display screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>*Refer to Status Cleaning Procedure for insert cleaning.</p> |
| <p>100. V.A.C. unit - Wound</p>  | <p>IH owned- Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. Remove disposable parts. <p>EVS</p> <ul style="list-style-type: none"> At discharge. After 30 days re-clean. | <p>Display screen Alcohol wipes</p> <p>All other parts AHP Wipes</p> <p>Follow EVS best practices.</p> <p>Attach green "I am clean" label and return to clean storage area.</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|--|---|--|
| <p>101. V.A.C. unit - Wound</p>  | <p>Vendor owned-Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> Remove disposable parts. Clean and place in designated vendor pick-up area. | <p>AHP wipes</p> |
| <p>102. Ventilators, Various models, (Bipap, CPAP, Optiflow)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> Remove disposable items. After each use. At discharge. | <p>Display screen Alcohol wipes</p> <p>All other parts AHP wipes</p> <p>Follow respiratory therapy best practices.</p> <p>Attach green "I am clean" label and return to clean storage.</p> |
| <p>103. Vital Signs Monitor, mobile</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> After each use. | <p>AHP wipes</p> |




[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|---------------------------------|--|--|
| <p>104. Warmer, blanket</p>  | <p>Various locations</p> | <p>EVS</p> <ul style="list-style-type: none"> • High touch exterior Surfaces daily. • Interior surfaces quarterly. • When soiled. • On unit request. | <p>Follow EVS best practices.</p> |
| <p>105. Warmer, blood/IV fluid</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • Remove all disposable tubing. • After each use. <p>EVS</p> <ul style="list-style-type: none"> • At discharge. | <p>AHP wipes</p> <p>Attach green “I am clean” label and return to clean storage.</p> <p>Follow EVS best practices.</p> |
| <p>106. Warmer, infant and neonatal</p>  | <p>Point of care</p> | <p>EVS</p> <ul style="list-style-type: none"> • High touch surfaces daily. • At discharge. • Between patients. | <p>Follow EVS best practices.</p> |


[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|---------------------------------|--|--|
| <p>107. Warmer, patient (Bair Hugger)</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. <p>EVS</p> <ul style="list-style-type: none"> • At discharge. • Between patients. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>108. Warmer, radiant overhead</p>  | <p>Point of care</p> | <p>HCP</p> <ul style="list-style-type: none"> • After each use. <p>EVS</p> <ul style="list-style-type: none"> • At discharge. • Between patients. | <p>AHP wipes</p> <p>Follow EVS best practices.</p> <p>Attach green “I am clean” label and return to clean storage.</p> |
| <p>109. Warmer, Sage Wipe</p>  | <p>Various locations</p> | <p>EVS</p> <ul style="list-style-type: none"> • High touch exterior surfaces daily. <p>HCP</p> <ul style="list-style-type: none"> • Interior surfaces when visibly soiled. | <p>Follow EVS best practices.</p> <p>AHP wipes</p> |

[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|---|--------------------------------|--|---|
| 110. Water/Ice Machine  | Various locations | EVS <ul style="list-style-type: none"> High touch exterior surfaces daily. FMO <ul style="list-style-type: none"> Interior every 6 months. | Follow EVS and FMO best practices. |
| 111. Wedges  | Point of care | HCP <ul style="list-style-type: none"> After each use. Weekly if patient dedicated. When soiled. EVS <ul style="list-style-type: none"> At discharge. Between patients. | AHP wipes Follow EVS best practices. Attach green "I am clean" label and return to clean storage. |
| 112. Wheelchairs and cushions  | Point of care | HCP <ul style="list-style-type: none"> After each use. Weekly if patient dedicated. When soiled. EVS <ul style="list-style-type: none"> At discharge. | AHP wipes Follow EVS best practices. Attach green "I am clean" label and return to clean storage. |

[Back to Index](#)

| Name of Equipment | Suggested Location of Cleaning | Responsible for Cleaning and Disinfecting and Frequency | Cleaning Instructions |
|--|--------------------------------|---|---|
| 113. Workstation on wheels (WoW) (e.g., Computers)  | Point of care | HCP <ul style="list-style-type: none"> Daily. If visibly soiled. After direct or indirect patient contact. | Display/screen Alcohol wipes All parts AHP wipes Clean display screen, keyboard, mouse, barcode readers, holder, and microphones. |

[Back to Index](#)

| | | | |
|-------------------|----------------------------------|------------------------------------|--|
| Effective Date | September 2024 | | |
| Last Reviewed | November 2025 | | |
| Approved By | IPAC | | |
| Partners Reviewed | Lab, EVS, Biomed | | |
| Owner | Infection Prevention and Control | | |
| Revision History | Date | Section | Revision |
| | November 5, 2025 | General Principles | Added Green tape, EVS Best practices – link to EVS standard operating procedures |
| | | Cleaning and Disinfection Products | Refer to HCP's use not EVS |

Interior Health would like to recognize and acknowledge the traditional, ancestral, and unceded territories of the Däkelh Dené, Ktunaxa, Nlaka'pamux, Secwépemc, St'át'imc, syilx, and Tšilhqot'in Nations where we live, learn, collaborate, and work together.