



Products

IH Facilities: Use Accel disinfectant wipes or 70 % isopropyl alcohol swabs
Non-IH Facilities: Use Clorox disinfecting wipes or 70% isopropyl alcohol swabs

Do not use bleach
Do not use sprays

Step 1 Clean

- Wipe device to remove foreign matter using gentle friction

Step 2 Disinfect

- Wipe with a fresh wipe to disinfectant
- Allow surfaces to remain wet for the necessary "contact time" as written on the product label
- Avoid ports (openings)

Step 3 (Optional) Rinse

- Wipe with a water dampened cloth after contact time is reached, to avoid damage to screens from cleaning residue

When to Clean and Disinfect Equipment

- Immediately after use (between clients)
- Routinely (as scheduled)
- When visibly soiled
- Before storage
- Before sending for repair

Cleaning Steps after Being in Isolation

1. Remove Personal Protective Equipment.
2. Perform hand hygiene and don clean gloves
3. Disinfect device per above
4. Remove gloves and perform hand hygiene

Virtual Care Team

Monday through Friday
7 a.m. to 4 p.m. (PT)
1-855-870-4755
VirtualCare@interiorhealth.ca

Vendor Information

[Apple](#), [AirPods](#), [Samsung](#)